Surgical Face Mask Product information and User Guide

Language: **English** 
Name of product: Surgical Face Mask, Non-Woven Face Mask
Type of product: Type IIR
Product description: 175 x 95 mm, Elastic Earloop, adjustable nosebridge, three layers.
CE EN 14683:2019+AC:2019
EU Directive 93/42/EEC on Medical Devices (MDD), Annex IX: Class I devices.

Product Information:

	Haian Medigauze Co., Ltd. Jiaoxie Industry Park, 226633, Haian, Jiangsu, China Tel+86-513-88240958 Mobile : +86 13901478762 Email: medigauze@medigauze.com www.medigauze.com
EC REP	Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany
Distributor in Europe:	VOVI AS, Sologårdsveien 71, 5243 Fana, Norway. sale@vollvik.com
	Nonsterile
	Single use only
Product description	Layer: 3 1st ply 25g/m ² spun-bond PP 2nd ply 25g/m ² melt blown 3rd ply 25g/m ² spun-bond PP
Quantity in packaging	50 pcs in Box
LOT	On packaging
	24 months after production date.
Storage or/and handling conditions	Should be stored in a clean, dry, ventilated environment, related humidity below 80%
	Use for invasive surgery, avoid splashing of fluids and liquids. If package or product is damaged, prohibit to use!
Warning:	If reused performance of the product may deteriorate, cross-contamination may occur!
Performance: (EN 14683:2019+AC:2019) Ref.: TÜV SÜD test report 721653351	Bacterial Filtration Efficiency (BFE): 99,5% Differential pressure: 39,1 Pa/cm ² Synthetic Blood Penetration: None seen Microbiological cleanliness: 3 CFU/g

Examples:

Soft and pleasant to skin
High comfort and easy to breath in
Splash resistance Type IIR classification

Warnings:


- Do not touch the outside of the facemask. If you do, wash your hands properly.
- This mask is only single use. Do not reuse.
- The mask should be replaced in time. Long-term use is not recommendable.
- It is suggested to stop using it if there is any unfitness or adverse reactions in the process of wearing
- The product is not washable, please ensure that it is used within the valid period.
- Store in a dry, ventilated place and stay away from fire and flammable materials
- Change mask if wet, heavy breathable, dirty or at least within (4) hours.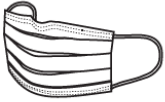

User guide and warning. Illustrated.

How to put on a face mask

1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Remove a mask from the box and make sure there are no obvious tears or holes in either side of the mask.
3. Determine which side of the mask is the top. The side of the mask that has a stiff bendable edge is the top and is meant to mold to the shape of your nose.
4. Determine which side of the mask is the front. The coloured side of the mask is usually the front and should face away from you, while the white side touches your face.
5. Face Mask with Ear loops: Hold the mask by the ear loops. Place a loop around each ear.
6. Pull the bottom of the mask over your mouth and chin.
7. Mold or pinch the stiff edge to the shape of your nose.

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How to remove a face mask

1. Clean your hands with soap and water or hand sanitizer before touching the mask. Avoid touching the front of the mask. The front of the mask is contaminated. Only touch the ear loops/ties/band. Follow the instructions below for the type of mask you are using.
2. Face Mask with Ear loops: Hold both ear loops and gently lift and remove the mask
3. Throw the mask in the trash. Clean your hands with soap and water or hand sanitizer

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EN 14683:2019+AC:2019

EU Directive 93/42/EEC on Medical Devices (MDD), Annex IX: Class I devices.

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